

DR-Go Quickguide

WARNING – This guide is not a substitute for the manuals; X-ray equipment is heavy and potentially dangerous. For your own SAFETY and to prevent damage read and understand the manuals first.

Normal Operation:

1. When turned on the unit will show the **(Ch)**arge of the **(CA)**pacitor as a percentage, then the **(Ch)**arge of the battery **(bt)** before being ready for use.
 - **TIP:** Save time by pre-charging the capacitor by turning it on for a few minutes at the beginning of the day, then when you need it, it may only take a few seconds to top up.
2. Set your exposure factors
3. Turn on the LBD either with the button on the control OR by double clicking to the 1st step of the exposure button.
 - **TIP:** LBD and lasers turn off automatically after an exposure or after 30 seconds.
4. Position your detector and patient
5. Ensure **READY** is displayed on the laptop, depress exposure switch to 1st (Prep) position. Ensure the **READY LED** is lit on the tube head.
6. Expose by depressing exposure button to 2nd position (the X-Ray LED will light up during exposure).

Troubleshooting:

- The unit will not expose
 - Has enough time been allowed for charging of the capacitor? If still charging the WAIT LED will be lit.
 - Ensure exposure switch is connected securely
- The display appears upside down.
 - Hit the R key to flip display back 180 degrees if it was pressed accidentally.